
JV35

clarity®

The Talking Phone™

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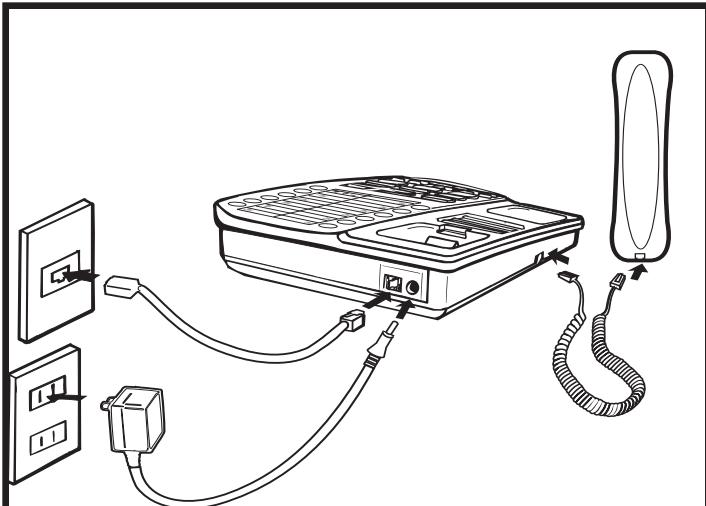Connecting your JV35
Comment Brancher votre JV35
Conexión de su JV35

This Quick Start Guide will help you get started with your new JV35. Please read the User Guide booklet to learn more about its advanced features.

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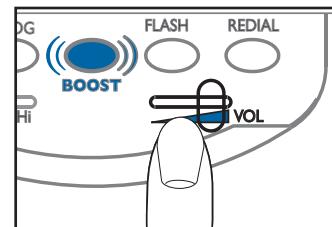
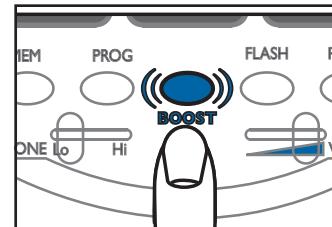
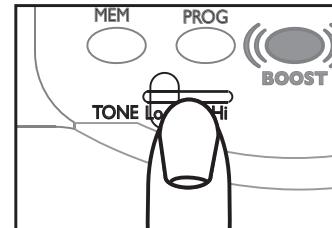
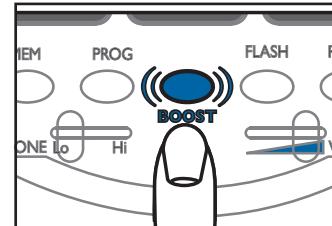
The next time and every time you press **BOOST**, the JV35 will give you the best listening clarity.

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
Programming Frequently Dialed Numbers

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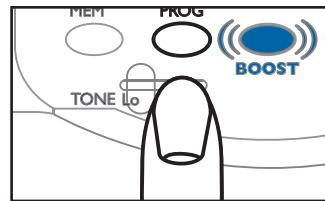2. Enter a phone number as you would dial it.

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5. The number is now stored within the button you have selected.

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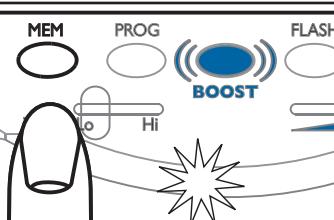
2. You will hear an announcement such as "Memory one" and then the number will be dialed.

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©2016 Clarity, a Division of Plantronics, Inc. All rights reserved. Amplifying Your Life, Clarity, JV35 and The Talking Phone are trademarks or registered trademarks of Plantronics, Inc.

Rev. B (08-16)

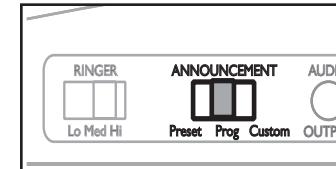
Recording Memory Button Identification Announcements

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1. Slide the ANNOUNCEMENT switch to PROG.

Pick up the handset. You will hear "Program Mode, Program Mode", followed by a two beeps.
Glissez le bouton ANNOUNCEMENT sur PROG.

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
2. Press MEM plus one of the number buttons.

After the "beep", record your announcement (i.e., George's office).

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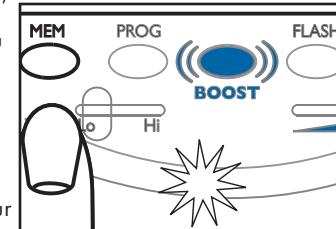3. A "beep" tells you recording has stopped. Your recording will play back immediately, followed by a "beep".

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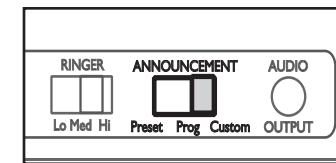
4. Repeat steps 3 and 4 to change your announcement or to record an announcement for a different memory button.

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6. Slide the ANNOUNCEMENT switch to Custom.

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